

## **Swift®** In-Bed Repositioning System

## The Facts:

The Swift® In-Bed Repositioning System features 2 elements, the Swift® Slider and the UltraSlide® Bottom Sheet, which work together to support repositioning & turning manoeuvres in

- COLOURS: White with blue / yellow (see specification)
- FABRIC: High performance Whootex jersey polyester & taffeta sliding panels. Ultraslide taffeta features carbon stripes
- HANDLES: Swift Slider with Handles features handles along the side to attach to a hoist
- FABRIC WEIGHT: Various
- SWL: 400KG (Standard & Long Length Sliders)

200KG (Slider with Handles)

- Multiple Use Slide Sheet Can be laundered for use with different individuals
- Class 1 Medical Device
- Instructions available

Laundering & Cleaning
• Swift® can be laundered following standard hospital laundry guidelines HSG(95)18. See instructions for














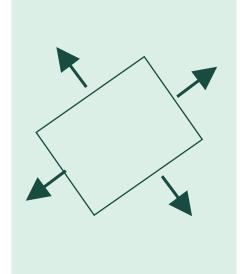





The Softer Side of Healthcare



PRODUCT GTIN

## Specifications:

COLOUR

CODE

Ultraslide® Bottom Sheet

PTD-LS/S/UK	White / Blue	Standard	5060540631046
PTD/LS/BA1/Y/UK	White / Blue	Bariatric	5060540631053
Swift® Sliders			
PTD-56/BA/Y/UK	White / Yellow	123 × 186 (Standard)	5060540631084
PTD-56/BA/LG/UK	White / Blue	200 × 205 (Long Length)	5060540631091
PTD-56/BA/H/UK	White / Yellow	200 × 243 (Handles)	5060540639332

SIZE

All codes are supplied individually

\* Manufacturing tolerances apply



Swift Slider - Standard and Long Length: DO NOT USE TO LIFT PATIENT

- Wash Swift® prior to first use
- Inspect for signs of wear and damage before use.
- Disposal always follow local guidelines.
- Contact manufacturer in case of serious incident.
- Consult your manual handling manager / guidelines before use

Swift Slider with Handles: DO NOT USE TO LIFT ALL OF THE PATIENT'S WEIGHT

- Wash Swift® prior to first use
- Inspect for signs of wear and damage before use.
- Disposal always follow local guidelines.
- Contact manufacturer in case of serious incident.
- Consult your manual handling manager / guidelines before use



